A&D
A&D Medical

Blood Pressure Log Book

Getting Started With Home Blood Pressure Monitoring

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Hypertension: The Silent Threat

You can't feel it — but it can change everything. High blood pressure quietly strains your heart and blood vessels every day.

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The Reality in Canada

- Almost 1 in 4 Canadian adults have hypertension.
- Rates rise sharply after age 60.
- Many Canadians don't know they have it.

The Good News

Early detection and small lifestyle changes can dramatically cut your risk. Prevention is about protecting yourself.

Measure. Understand. Act.

Experience clinical grade accuracy, tools designed for every lifestyle, and smart tracking that helps you live better and longer.

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Source : Cœur + AVC 2022

Why Home Monitoring?

Because your blood pressure changes throughout the day — and the clinic only captures one moment. Home monitoring gives a true picture of your heart health, helps detect hypertension earlier, and tracks how well your treatment is working.

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Why choose A&D Medical for home health monitoring?

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Hypertension
CANADA

At A&D Medical, we bring over 50 years of expertise in precision measurement in the professional setting and at home.

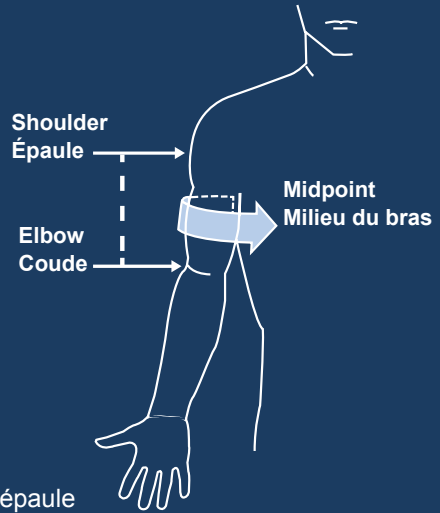
Clinically tested and trusted, our monitors appear on Hypertension Canada's list of reliable home blood pressure monitors-making it easier for accurate self-monitoring and better blood pressure management.

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
Cuff size matters

- Use the right size cuff
- Allow your arm to hang at your side
- Locate the midpoint between your elbow and shoulder
- Measure the circumference

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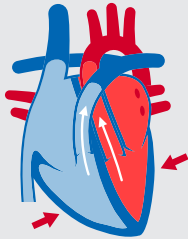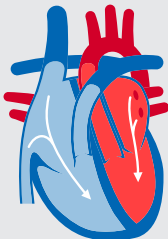
Diastolic Blood Pressure
La pression artérielle diastolique

What do my measurements mean?

Blood pressure is the measurement of the amount of power (pressure) that your blood is exerting on your artery walls.

Systolic (top number) shows how much pressure is exerted while your heart is beating. Diastolic (bottom number) is a measurement of the pressure between beats.

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Source: Hypertension Canada Guidelines 2025

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Before your measurement:

For 30 minutes prior to taking your blood pressure

- Do not exercise
- Do not drink coffee, caffeinated soda or alcohol
- Do not smoke
- Sit quietly for 5-10 minutes
- Ensure you are using the right size cuff

During your measurement:

- Do not talk
- Sit with your back straight and supported
- Uncross your legs and place feet flat on floor
- Rest arm on a table so that the cuff is at heart level
- Measure two times a day, in the morning and evening

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Your health in your hand

The A&D Heart Track App is the ultimate health management app designed by hypertension experts to empower you on your health and wellness journey. You'll discover a seamless experience that puts your health metrics at your fingertips.

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With a Bluetooth-enabled device, your blood pressure readings are seamlessly recorded and can be easily exported, making health tracking simple and stress-free.

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